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Instructions for use Specimen container with isotonic saline solution applies to the products specimen containers with saline solution in various designs

Product description/Intended use

The products Specimen containers with isotonic saline solution are used for transporting viral swab specimens. The products are specimen containers into which swabs can be placed after a smear has been performed, stored or transported until laboratory analysis, and further processed in the laboratory. The specimen containers, which are pre-dosed with isotonic saline, are IVD products intended for the detection of a viral load (e.g. SARS-Cov-2) in patients of any age and sex. Important information about the patient's health status can be obtained from the sample.

All products from the product group Specimen containers with isotonic saline solution are intended exclusively for single use in a professional environment by medically qualified personnel.

Features

The specimen container with isotonic saline solution is a CE-marked IVD product, which is grouped in class A. The tubes are pre-dosed with different amounts of 0.9% isotonic NaCl solution for transporting swab specimens.

Notes

- Wear protective gloves and, if necessary, other protective clothing!
- Do not use damaged or already used specimen containers.
- Any violent handling of the product must be avoided.
- After removing the lid, avoid unnecessary, sudden movements.
- Use specimen containers with saline solution only within the specified shelf life.
- The further storage or disposal of the used or contaminated specimen containers with saline solution is carried out according to the regulations of the facility in the suitable disposal container.
- The product is for single use only!
- The pre-dosed isotonic saline solution is intended to remain in the tube.
- The saline medium is **not** suitable for oral use of any kind (gargling, rinsing, moistening, etc.).

Application

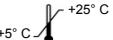
Open the lid of the container and place the swab in the container so that the lower part of the swab is completely covered by the solution, shorten the swab to a suitable length if necessary. Close the container again. Further sample processing steps are not part of these instructions for use.

Storage

Store dry at +5°C to 25°C and protect from direct sunlight and humidity.

All serious incidents related to the product shall be reported to the manufacturer and the responsible authority of the member state where the user and/or patient is registered.

Symbol meaning

	Item no.		Protect from sunlight
	Lot in YYX0000		Protect from rain
	Shelf life in YYYY-MM		IVD product
	CE-mark		Storage temperature
	Manufacturer		Follow information documents
	For single use only		

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KABE LABORTECHNIK GmbH