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## Instructions for use Specimen container for pathological/histological examinations valid for the products Specimen containers prefilled with 4% formaldehyde solution in different sizes

### Intended use

The specimen containers for pathological/histological examinations prefilled with 4% formaldehyde solution are used to hold pathological/histological specimens from the human body. The products are sterile specimen containers into which the tissue samples can be placed, prepared for laboratory analysis and further processed in the laboratory. Important information about the patient's health can be obtained from the pathological/histological specimen.

All products from the product group Specimen containers for pathological/histological examinations are intended exclusively for single use in a professional environment by qualified medical personnel.

### Features

The specimen container for pathological/histological examinations is a CE-marked IVD product, which is grouped in class A. The tubes are made of polypropylene, the lids of polyethylene and are pre-dosed with different amounts of 4% formaldehyde solution as fixative for pathological and histological samples, depending on the filling volume.

### Notes

- Obtain special instructions before use.
- Wear protective gloves and, if necessary, other protective clothing!
- Do not use damaged or already used specimen containers for pathological/histological examinations.
- Any violent handling of the product must be avoided.
- After removing the lid, avoid unnecessary, sudden movements.
- Use specimen container for pathological/histological examinations only within the specified shelf life.
- The further storage or disposal of the used or contaminated specimen container for pathological/histological examinations is carried out according to the regulations of the facility in the suitable disposal container.
- The product is for single use only!
- The medium formaldehyde is **not** suitable for consumption!
- In case of contact with skin: wash with plenty of water and soap.
- In case of exposure or if affected: Seek medical advice/attention.
- Use only at well-ventilated workplaces or with appropriate exhaust ventilation (AgW acc. to TRGS 900: 0.37mg/m<sup>3</sup>, 0.3 ppm; exceedance factor (short-term value) = 2, due to sensory irritation).
- Further information can be found in the safety data sheet, which is available on request.

### Hazard information

- Formaldehyde solution:
  - may cause cancer
  - can probably cause genetic defects
  - is harmful if swallowed
  - may cause allergic skin reactions

### Application

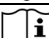
Open the lid of the container and place the sample to be analyzed in the formaldehyde solution with the appropriate auxiliary equipment so that the sample is completely immersed in the solution (the container must not be filled to the top). Close the container again. Further sample processing steps are not part of these instructions for use.

### Storage

Store dry at +5°C to 25°C and protect from direct sunlight and humidity.

**All serious incidents related to the product shall be reported to the manufacturer and the responsible authority of the member state where the user and/or patient is registered.**

### Symbol meaning

	Item no.		Protect from sunlight
	Lot in YYX0000		Protect from rain
	Shelf life in YYYY-MM		IVD product
	CE-mark		Storage temperature
	Manufacturer		Follow information documents
	For single use only		

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KABE LABORTECHNIK GmbH