# Gebrauchsanweisung URIVETTE BAK

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GA\_URIVETTE\_BAK\_d\_Rev\_06\_25

# Instruction for use URIVETTE BAK

## apply for the product URIVETTE BAK 10 ml

### Intended use

The URIVETTE BAK is designed for the collection, preparation and subsequent storage of urine samples for laboratory analysis. Using the included suction tube, the urine sample can, for example, be drawn from a urine cup. The samples can be safely transported in the tube, tested with test strips and centrifuged. The URIVETTE BAK is intended for use in patients of all ages and genders. Various parameters can be measured from the urine sample, providing essential information about the patient's health status. The URIVETTE BAK is prepared in such a way that it stabilizes the bacterial count in the urine for approximately 24 hours. It is intended for single-use only in a professional setting by medically qualified personnel.

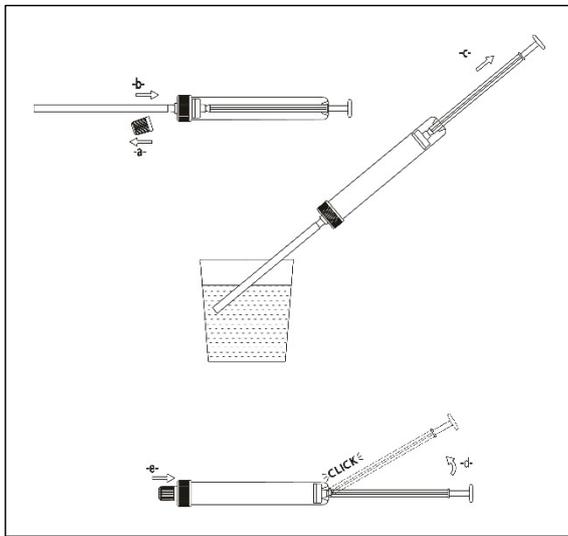
### Features

The URIVETTE BAK is a CE-marked IVD product classified under IVD risk class A. It consists of an aspiration tube, a green screw cap with cone, a cone cap, and a suction tube. A boric acid solution is pre-dosed as preservative.

### Notes

- Wear protective gloves and, if necessary, additional protective clothing!
- Use the product only within the specified expiration period.
- Avoid any forceful handling of the product.
- The product is intended exclusively for the collection of an urine sample from an urine collection container.
- For single use only!

### Application

- 1) Remove the cone cap from the cone of the URIVETTE BAK and keep it ready for resealing (-a-).
- 2) Attach the suction tube to the cone (-b-).
- 3) Withdraw the urine from the collection container by pulling back the plunger. Ensure you stay above the container to avoid potential contamination (-c-).
- 4) **NOTE:** After filling the URIVETTE BAK, residual urine may remain in the suction tube. To avoid contaminating the user or work environment, hold the URIVETTE BAK with the suction tube pointing downward and remove it above a drain or the urine container. Discard it afterwards.
- 5) Break off the plunger rod (-d-).
- 6) Close the URIVETTE BAK by placing the cone cap on it (-e-).
- 7) **Test strip test:** Open the URIVETTE BAK and perform the test strip test directly inside the tube. The length of the URIVETTE is tailored to the test strip. **Sediment analysis:** For sediment analysis, centrifuge the URIVETTE (approx. 5 minutes at 400–800 g). During centrifugation, the sediment settles at the bottom of the plunger.

### Storage

Store in a dry place at 5°C to 25°C and protect from direct sunlight and humidity.

**All serious incidents related to the product shall be reported to the manufacturer and the responsible authority of the member state where the user and/or patient is registered.**

### Symbol meaning

	Item number		Protect from sunlight
	Lot in YYX0000		Store in a dry place
	Can be used until YYYY-MM		IVD product
	CE marking		Do not use if the packaging is damaged
	Manufacturer		Follow the instructions for use
	For single use only		Temperature limits
	Unique Device Identification		

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KABE LABORTECHNIK GmbH